FFP2 NR

effimed 

MASCARILLA AUTOFILTRANTE EFFIMED MSK-EFF-EP2

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CTRA CÓRDOBA-MÁLAGA, km. 80,800

14900 – Lucena – Córdoba – España

**ONE-USE FOLDABLE AUTOFILTERING
MASK TO PROTECT FROM PARTICLES**

TRADEMARK:	 effimed
REFERENCE/MODEL:	MSK-EFF-EP2
CLASS AND TYPE:	FFP2 NR

STANDARD:	EN149:2001 + A1:2009 Regulation (EU) 2016/425
"EC" MARKING:	 0161
ASOCIACION DE INVESTIGACION DE LA INDUSTRIA TEXTIL (AITEX): nº 0161	Plaza Emilio Sala, 1 03801 Alcoy (Alicante) Phone: +34 96 5542200 Email: rmunoz@aitex.es www.aitex.es

AUTOFILTERING FFP2 EFFIMED MSK-EFF-EP2 MASK

This one-use disposable mask belongs to the FFP2 category in accordance with standard EN 149:2001+A1:2009. The appropriateness for use and identification of class is on the container and non-sterile half-mask. It is suitable for protecting users against inhalation of particles and polluting agents, spread through dust and solid and liquid aerosols. It prevents polluting particles from

entering the body through the respiratory tract, by filtering inhaled air.

SIZE:	21,0x15,7 (\pm 0,5 cm)
VALIDITY:	Three years from manufacturing
MATERIAL:	100% non-woven fabrics
MANUFACTURER:	Efficold, s.a.
MANUFACTURER'S ADDRESS:	Carretera Córdoba – Málaga, km 80.800 14900 Lucena (Córdoba)

CONTROLS BEFORE USE

- » Before use, check the product is within its useful life.
- » Wash your hands with soap and water or rub them with a hydro alcoholic solution before handling the mask.

GUIDELINES TO ADJUST AND PLACE THE MASK:

1. Place the respirator against the chin, with the nose clip upwards.
2. Keep the mask over your nose and mouth.
3. Holding the mask with your hand, pull the elastic bands over the back of the ears.
4. Using both hands, adapt the adjustable extender hook for the mask strap and bend the nose clip in order to ensure a proper facial sealing.

5. Test to control the correct placing of mask without a valve: Place your hands on the mask and breathe out vigorously. If you notice leaks inwards/outwards, adjust it again, placing the half-mask over your face again and, if necessary, pressing the nose clip and/or using an adjustable extender hook for the mask strap.

6. Avoid touching the mask during use. If you do, wash your hands.

7. Replace it if it gets wet or following 12 hours of use.

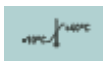
8. Replace the mask immediately if you notice breathing trouble, if the mask is faulty or it does not adhere well to the face. Carefully following these guidelines is essential for the safe use of the respirator.

9. Remove it from the back, don't touch the front part of the mask.

USE LIMITATIONS

- » This product does not provide oxygen. Use only in well-ventilated areas with enough concentration of oxygen to preserve life. Limits referring to the presence of oxygen in an environment for the use of these masks may vary from one country to another. Therefore, it is recommended you follow the parameters defined by the country of use as a reference. EFFICOLD, S.A. does not recommend use in any environment whose concentrations of oxygen are below 19.5%.
- » Do not use in environments where there is a risk of explosion.

MARKED

Temperature range

Maximum relative humidity

Recycling advice

Not reusable

Name and address of the manufacturer

Indication that the product should not be used if its packaging has been damaged or has been opened.

Manufacturing batch matches the manufacturing date. (YYYY / MM)

Expiration date (YYYY / MM)

ATTENTION: [PPE – Personal Protective Equipment]

- » If it is impossible to make the mask adhere adequately to the face's shape, do not enter environments with potential risks.
- » A breach of these provisions may result in incorrect use of the PPE and cause diseases and death.
- » The chosen respirator with the adequate shape is essential to protect your health. Before using the PPE, check with a person trained in Industrial Hygiene or experts in Occupational Health and Safety to determine the adequacy of use.
- » Abandon the work area immediately (A) if breathing becomes difficult or (B) if you feel anxiety or dizziness.

- » Beard hairs and breathing trouble may reduce the efficiency of these filtering half-masks.
- » Not appropriate for users with damaged ears.
- » Do not change or alternate the filtering half-mask under any circumstances.
- » One use only: No maintenance required.
- » Dispose of the half-mask after one use.
- » Keep unused half-masks in their container intact away from sunlight, in dry unpolluted environments (temperature between 10°C and + 40°C and relative humidity below 80%).
- » The identification code of the model is located on the product's container.

**PERFORMANCE AUTOFILTERING FFP2
EFFIMED MSK-EFF-EP2 MASK**

FFP2 NR WITHOUT VALVE Ref. MSK-EFF-EP2 Trademark EFFIMED Standard EN 149:2001+A1:2009	Perfor mance FFP2
Penetration of filtering material (NaCl 95 l/min)	< 6%
Penetration of filtering material (paraffin 95 l/min)	< 6%
CO2 contained in inhaled air	<1%

Resistance to breathing: inhaling (30 l/min) mbar	<0,7
Resistance to breathing: inhaling (95 l/min) mbar	<2.4
Resistance to breathing: exhaling (160 l/min) mbar	<3
Total leak inwards	< 8%

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EFFICOLD, S.A. rät jedoch vom Gebrauch in Bereichen mit einer Sauerstoffkonzentrationen, die unter 19,5% liegt, ab.
HERGESTELLT VONEFFICOLD S.A.